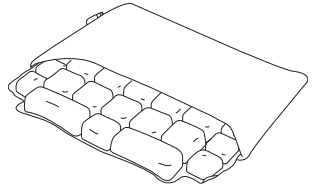
# ROHO® LTV® Seat Cushion Owner's Manual

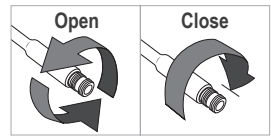
Package contents: LTV cushion, LTV cover, repair kit, hand pump, owner's manual, limited warranty supplement, product registration card

**⚠ Warnings:** DO NOT use the cushion as a water flotation device (e.g., a Life Preserver). It will NOT support you in water. Keep your cushion away from sharp objects and hot ashes.

## To Adjust Your LTV Seat:

1. Open the valve. Inflate with the hand pump until all of the air cells are visibly filled. Close the valve and then remove the hand pump. Make sure the inflation valve is closed.

*Note: To open, turn the inflation valve at least one full rotation.*

2. Place the LTV Seat on your sitting surface: The non-skid side of the LTV Cover should face down, and the zippered portion of the cover should face the back. The valve should be on the front left corner.
3. Sit on the LTV Seat in your normal sitting position.
4. Open the valve to release air. As air is released, you will slowly sink into the LTV Seat. Release air until you get close to your desired comfort level, and then close the valve. Release air or add more air until you reach your desired comfort level.

## To clean the LTV Cover:

Remove the cover from the cushion. If the cover is Quilted Fabric, machine wash cold, gentle cycle, with like colors. Air dry. If the cover is ULTRALEATHER®, hand wash with mild detergent. Wipe or air dry.

## To clean the LTV Cushion:

Close the valve and place the cushion in a large basin. Use a liquid dishwashing detergent, laundry detergent, or another multipurpose detergent that is used for general cleaning. Follow the surface-cleaning directions on the product label. Use a soft plastic bristle brush, sponge, or cloth to gently scrub all surfaces. Rinse the cushion with clean water and air dry or towel dry.

**⚠ Cautions:** DO NOT expose to direct sunlight. • DO NOT use automatic dishwasher detergents, which are too caustic for personal safety.

*Note: Deflated air cells may be easier to thoroughly clean. To deflate the air cells: with the inflation valve open, roll up and/or compress the cushion, and then completely close the inflation valve.*

## LTV Seat Limited Warranty

What is covered? Defects in materials or workmanship of the product.

For how long? 12 months from the date the product was originally purchased, with the exception of the cover, which has a 6 month Limited Warranty.

What is not covered? A product that has been misused, damaged by an accident, or damaged from "an act of Nature," e.g., flood, tornado, earthquake, fire. Any substitutions of supplied product accessories or modification to the product will void the warranty.

What the customer must do: Contact Customer Support. Items sent without prior authorization will not be returned to the customer.

Disclaimer and Limitation of Warranties: Any implied warranties, including the MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, are limited to the time period defined in the instructions provided with the product, and any action for a breach of such warranties or any express warranties herein must be commenced within that time. Permobil shall not be responsible for any incidental or consequential damages, whether as a result of a breach of express or implied warranties or otherwise. Some jurisdictions do not allow limitations on how long implied warranties will last and some jurisdictions do not allow the exclusion or limitation of consequential or incidental damages, so those provisions may not apply to you. While this warranty gives you specific legal rights, you may also have other rights which vary from jurisdiction to jurisdiction. Any warranties and services are limited to products purchased from authorized Dealers in the USA or International Distributors. If you purchase a ROHO product from an unauthorized Dealer or an unauthorized International Distributor, or if you reside outside the USA and you purchase a ROHO product from anyone other than the authorized International Distributor for your country of residence, your warranty will not be valid.

Return Policy: Product must be new and unused, and prior authorization is required; contact Customer Support. May be subject to a restocking charge.

---

<b>Contact Information</b>	permobil.com
In the U.S.A.	Customer Support: orders.roho@permobil.com Phone: 800-736-0925 • Fax: 888-551-3449 ROHO, Inc., 1501 South 74th Street, Belleville, IL 62223 U.S.A.
Outside the U.S.A:	intlorders.roho@permobil.com Phone: 618-277-9150 • Fax: 618-277-6518
If directed to return any items, please send to:	ROHO, Inc. 1501 South 74th Street Belleville, IL 62223 U.S.A.

---

 ROHO, Inc., 1501 South 74th St., Belleville, Illinois 62223 U.S.A.  
LTV Cushion made in the U.S.A.

Products may be covered by one or more U.S. and foreign patents and trademarks, including LTV®, LTV Seat®, ROHO®, and shape fitting technology®. ULTRALEATHER® is a registered trademark of Ultrafabrics.

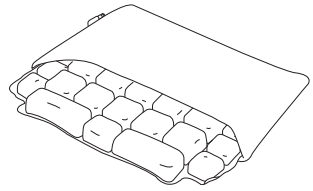
Product information is changed as needed; current product information is available at permobil.com.

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© 2005, 2022 Permobil